


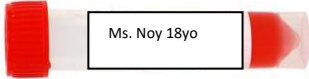







Specimen triple-packing instruction flowchart

		
<p>1. Proper labeling with patient ID (Name and age)</p>	<p>2. Secure primary Parafilm and/or absorbent tissue</p>	<p>3. Place specimen tube in individual Ziplock</p>
		
<p>4. Seal zip-lock bag</p>	<p>5. Place in a durable sturdy and leak proof plastic box or Biohazard plastic bag (Well tined with tape)</p>	<p>6. Place the specimen box in Cool box with ice packs (two ice-pads should be placed) on both sides of the container</p>
		
<p>7. Disinfect outer Cool box with 0,1 % sodium hypochlorite or Alcohol 70% solution</p>	<p>8. Tightly seal outer container and label the third container</p>	<p>9. Attach label together with the laboratory request form (inside zip lock) on top of the outer container.</p>